

General Information

- May impart more of a feeling of security around the edges of your skin barrier
- May help reduce skin barrier edge lifting
- May help increase skin barrier wear time
- Easily fits around the edges of your skin barrier without adhesive border barrier
- Conforms to your body's folds and creases
- Provides extra adhesion to accommodate a parastomal hernia



Box of 30

Using Adapt Barrier Extenders

1. Prepare the skin by removing existing skin barrier and clean the skin around the stoma. Apply new skin barrier / pouching system
2. Remove the paper backing from the barrier extender by stretching lightly at each end of the extender
3. Release only one side of the paper backing
4. Place the adhesive end of the barrier extender along the perimeter of the skin barrier; covering both skin barrier and the skin next to it. Remove the remaining paper backing. Run your fingers along the barrier extender to ensure that it adheres well
5. Repeat for any additional strips you require around your barrier
6. The barrier extenders will remove easily with your routine pouch change



1



2



3



4



5



6

Adapt Barrier Extenders

Frequently Asked Questions

Q: Can I use barrier extenders with skin barriers that have an adhesive border barrier?

A: YES. Hollister is committed to helping you achieve a good fit with your pouching system. A tape border skin barrier is already designed to provide added security and prevent edge lifting. A barrier extender can be used to help provide extra security when necessary and may increase wear time.

Q: Will this product provide security with my pouching system?

A: YES. This product will help secure the skin barrier and helps reduce edge lifting. If you are experiencing leakage problems, you should seek professional assistance from a stoma care nurse or health care professional.

Q: Do barrier extenders contain latex?

A: NO. The product is made with hydrocolloid adhesive and polymer/copolymer plastics and not with natural rubber latex.

Q: Is the packaging recyclable?

A: YES. Hollister strives to be environmentally responsible by being a steward of earth's limited resources. The plastic packaging is recyclable.

Q: Can I use less than three barrier extenders?

A: YES. The strips are to help provide you with extra security around your barrier. Use the strips in areas which you feel necessary or contact your stomal therapy nurse or health care professional to discuss.

Additional Instructions:

For product questions, sampling needs, or detailed clinical questions concerning our products call:

Australia 1800 880 851

New Zealand 0800 678 669

Routine follow-up with your healthcare professional is recommended.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

Not all products are CE marked.

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2020 Hollister Incorporated. AUH207. March 2020.



Australia
PO Box 375
Box Hill, VIC 3128
1800 880 851
www.hollister.com.au

New Zealand
PO Box 62-027
Mt Wellington, Auckland
0800 678 669
www.hollister.co.nz