

Advance

Touch Free Intermittent Catheter



Confidence. Pure and Simple.

Ultra-smooth eyelets
For gentle insertion and removal

Touch free sleeve
Helps you freely handle the catheter

Gel reservoir
Controls catheter lubrication

Protective tip
Helps the catheter bypass
environmental bacteria



The Advance intermittent catheter is a hygienic, pre-lubricated gel catheter that helps you manage your daily routine with confidence. Recommended by your clinician, it is designed to help you catheterise easily.

People First:

Ria Bakker, The Netherlands

"Strange things happen when you are disabled. You lose friends, your boss chooses not to understand, people stare, or pretend not to notice. But with that comes new friendships, a better job, and, well, when people stare, I stare back."






Remove ring cap from protective tip of catheter.



Hold the gel reservoir in one hand and with the other hand move the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.



Spread the labia and cleanse around opening of the urethra with mild, unscented soap or a non-alcoholic wet wipe.

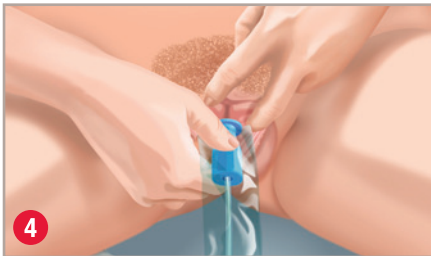

Holding the gel reservoir gently, insert the protective tip until the base comes in contact with the urethral opening. Keep the protective tip in place. Release the labia.



Continue to insert the catheter forward until the urine starts to flow. Once urine has stopped flowing, slowly withdraw the catheter.



Dispose of catheter and package.

