


Adapt Barrier Rings

General Information

- Use to protect your skin
- Use to fill in uneven surfaces to create a flatter surface for a more secure fit for your pouch
- Use to extend the wear time of your pouch
- Use to help prevent ostomy drainage from getting under the ostomy barrier
- Can be cut, stretched, or stacked together to improve the fit of your pouching system

Using Adapt Barrier Rings

1. Wash and dry your skin thoroughly
2. Remove the barrier ring from the plastic container (Figure 1), then remove the protective coverings from both sides of the barrier ring (Figure 2)
3. Stretch and shape before applying (Figure 3). You may choose to:
 - a. Apply the barrier ring to the adhesive side of your skin barrier (Figure 4)
 - b. Apply the barrier ring to your skin and then apply your pouching system
4. Stack barrier rings if needed, for an improved fit (Figure 5)
5. To remove, gently peel away from the skin when you change your pouching system

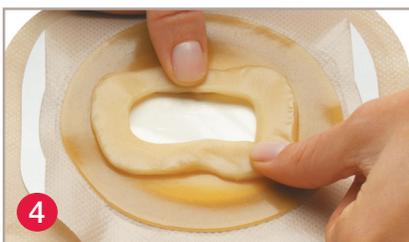


Adapt Barrier Rings are available in the following sizes:



Adapt Barrier Rings

Stock No	Outer diameter	Size
7805	48 mm	Small
7815	48 mm Slim	Small
7806	98 mm	Large

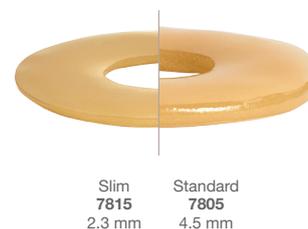

Frequently Asked Questions

Why would I choose to use a slim ring?

The Adapt Slim barrier ring is designed to be stretched and shaped like the standard version but offers a lower profile.

Does it matter which side of the ring is applied to my skin

No, either side may be applied to your skin.

My stoma is not round. Can I still use the barrier rings?

Yes, Adapt barrier rings can be easily stretched or shaped for a customised fit.

I used to use paste. Are the barrier rings better than paste?

There are many advantages to using Adapt barrier rings instead of paste. For some people, barrier rings are easier to apply and remove than paste. Barrier rings are more durable and tend to be more resistant to erosion from your ostomy discharge.

Most people find that the barrier rings eliminate the need for paste.

Can I use this if my skin is irritated?

Yes*, Adapt barrier rings are alcohol-free, so there is no sting from alcohol when applied to broken or irritated skin.

Can I obtain the Adapt barrier rings on the Australian Stoma Appliance Scheme (SAS)? What about in New Zealand?

Yes. All adapt barrier rings are available on the Australian SAS. The Adapt barrier rings are also available in NZ.

What is the advantages of the clam shell packaging?

The clam shell design packaging allows you to easily store any part of the Adapt barrier ring you haven't used for use at a later time. It also allows you to easily see the remaining amount of Adapt barrier ring at a glance. The clam shell is also stackable for convenient storage and is environmentally friendly, recyclable and made from recycled materials. ♻️

Is the formulation the same as the standard barrier ring?

All Adapt rings are made from the same hydrocolloid formulation. This formulation is flexible and provides erosion resistant extended wear benefits, to help protect your skin.

For more information or samples please call Customer Engagement on:

Australia **1800 219 179**
New Zealand **0800 678 669**

www.hollister.com