


Product Code: 3778

Collect Supplies

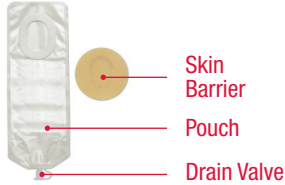

Gather pouch, skin barrier, measuring guide, scissors, pen, water, and gauze, cotton balls, or tissue.

Prepare the Skin Barrier and Pouch



1 Measure the stoma using the measuring guide or use a pattern provided by your nurse.



2 Trace the pattern onto the paper side of the skin barrier. The skin barrier should fit closely around the stoma.



3 Using small curved scissors, carefully cut an opening in the skin barrier to match the pattern. Do not cut beyond the line on the release liner.



4 Remove the release liner from pouch.



5 With the paper side of the skin barrier facing up, centre the skin barrier on the adhesive of the pouch. Hold together for 10 seconds. No adhesive should be visible beyond the skin barrier.



6 Remove the release liner from the skin barrier. Set aside with adhesive facing up. Close drain valve at bottom of pouch by attaching cap.



7 Carefully lift an edge of the skin barrier and peel downward, gently pushing the skin away from the skin barrier.

Properly dispose of used pouch. Do not flush down toilet.

Remove Used Pouching System

Clean and Dry Skin



8 Clean the skin with water or with a gentle soap as recommended by your healthcare provider. Rinse and dry thoroughly. Do not use oils, baby powder, lotion, or lotion-type soaps on the skin.

Apply the New Pouching System



9 Centre the skin barrier opening over the stoma. Gently press the barrier against the skin until the entire barrier adheres.

Tips

- To connect the newborn pouch to tubing, detach cap from bottom of drain valve. Insert drain tube into drain valve and connect other end of tubing to a bedside receptacle
- The newborn pouch has a spout at the bottom. If the discharge becomes too thick, you may cut off the bottom of the pouch at desired location. Cut through the plastic pouch stabilisers located inside the pouch and remove them
- Apply the closure included in the box. Refer to detailed instructions inside box. If too much is cut off the pouch capacity will be diminished
- Empty pouch when 1/3 full. If the capacity is insufficient, consider a larger paediatric pouch
- The skin around the stoma should not be irritated or broken. If the skin is irritated, contact your STN

For more information, see product instructions.

Newborn Pouching System

Additional Instructions

Additional Information

- **Caring for Your Child with an Ostomy** (an educational booklet for parents and caregivers of infants with ostomies)



Visit the Hollister website to download the booklet
www.hollister.com.au/en-au/ostomycare/ostomycalearningcenter/ostomypediatric

For product questions, sampling needs, or detailed clinical questions concerning our products, call Customer Engagement Australia **1800 880 851** or Customer Service New Zealand **0800 678 669**.

Routine follow-up with your healthcare professional is recommended.
Prior to using any ostomy products/accessories be sure to read all product inserts and labels for any indications, contraindications, hazards, warnings, cautions and directions for use.



Hollister Australia
PO Box 375
Box Hill, VIC 3128
1800 880 851
www.hollister.com.au

Hollister New Zealand
PO Box 62-027
Mt Wellington, Auckland
0800 678 669
www.hollister.co.nz