

ADAPT Lubricating Deodorant

78500, 78501

General Information

- Adapt Lubricating Deodorant reduces odours, eases emptying and helps prevent pouch static and sticking.
- It is not a perfume or masking agent.
- Adapt Lubricating Deodorant should be used in **drainable** and **closed** pouching systems only.

For Drainable Pouches

1. Apply pouch.
2. **Bottle:** add approximately 5ml or 1 tsp. (Figure 1).
Sachet: add the complete contents of the packet. (Figure 2).
Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).
Repeat Steps 1 or 2 after every emptying.

For Closed Pouches



1. Do not remove release paper on barrier to add product.
2. **Bottle:** add approximately 5ml or 1tsp. (Figure 4).
Sachet: add the complete contents of the packet.
3. Apply your pouch. Rub to thoroughly coat the inside of the pouch (Figure 3).

Note: Avoid contact with the adhesive on the skin barrier or flange. For use in ostomy pouches only.



78500 78501

Adapt Lubricating Deodorant

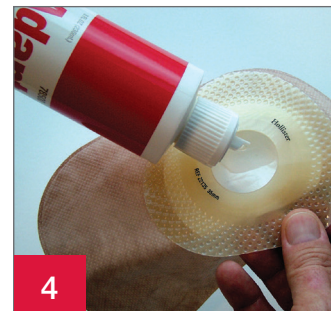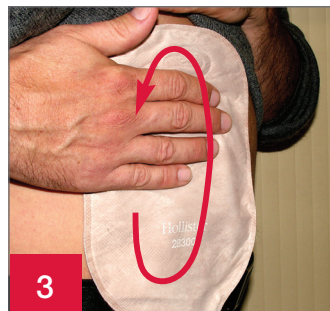

ADAPT Lubricating Deodorant

Frequently Asked Questions

Q: Does it matter if the Adapt Lubricating Deodorant comes in contact with the filter on the pouch?

A: When coating the inside of the pouch, it is best to avoid the filter.

Q: Do I need to use the Adapt Lubricating Deodorant every time I empty my pouch?

A: Yes. For the product to work effectively, it must be reinserted each time your pouch is drained.

Q: Can I just put a few drops in the bottom of my pouch and not rub it against the sides of the pouch?

A: No, this is not recommended since it will not be effective as a pouch lubricant or as a pouch deodorant. Rubbing the pouch helps to coat the inside of the pouch film.

Q: Wouldn't it be easier and cheaper to use a cooking spray instead to coat the inside of my pouch?

A: This is not recommended because cooking oil:

1. may cause the seal of your pouch to fail
2. does not reduce odour

In addition, Adapt Lubricating Deodorant is on the Australian Stomal Appliance Scheme / is reimbursed by many insurance companies as a pouch deodorant which makes it more cost effective.

Q: It doesn't have a scent. How do I know it is working?

A: Adapt Lubricating Deodorant is not a perfume or a fragrance used to mask or cover-up odours. It does not have any odour by itself, but it will trap odours in the pouch.

Q: Can the Adapt Lubricating Deodorant come in contact with the stoma?

A: Yes, it has been tested and it will not harm your stoma.

Q: What if Adapt Lubricating Deodorant comes in contact with my skin?

A: This product is not a skin irritant. It is water soluble so it will easily be removed from the skin with a moist cloth. It may cause issues with barrier adhesion so is advisable to clean the skin.

Additional Instructions:

For product questions, sampling needs, or detailed clinical questions concerning our products call:

Australia 1800 880 851

New Zealand 0800 678 669

Australia

PO Box 375
Box Hill, VIC 3128
1800 880 851
www.hollister.com.au

New Zealand

PO Box 62-027
Mt Wellington, Auckland
0800 678 669
www.hollister.co.nz

Routine follow-up with your healthcare professional is recommended.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2022 Hollister Incorporated. AUH224. July 2022.