

ADAPT Medical Adhesive Remover Spray/ ADAPT Universal Remover Wipe

7737

7760

General Information

- Helps remove adhesive skin barriers and/or tape
- Removes adhesive residue from the skin
- Silicone-based formulation
- No sting from alcohol

Using Adhesive Remover Spray

1. Shake well before using.
2. Peel back one edge of the skin barrier or tape border.
3. Hold the can so that the arrow faces the base of the skin barrier or tape, and spray until the adhesive releases. The 360° spray technology minimises waste, and will allow you to spray from any angle.
4. Ensure your skin is dry and apply your new ostomy pouching system as usual.

Using Universal Remover Wipes

1. Tear open the package and take out the wipe.
2. Press or rub the wipe against your skin as you peel off your skin barrier and/or tape.
3. Press or rub the wipe over any areas of the skin where tape residue remains.
4. Ensure your skin is dry and apply your new ostomy pouching system as usual.

Frequently Asked Questions

Q: Is an adhesive remover always needed?

A: No, this product is optional. Use it to remove adhesive residue from your skin, or to make it easier to remove your skin barrier or tape.

Q: Will this be uncomfortable to use?

A: This silicone-based product is formulated to be a no-sting alternative. When used correctly, the adhesive remover will help to make the removal of your pouching system easier and help reduce the potential for skin stripping.

Q: Do I need to wash the adhesive remover off of my skin?

A: The adhesive remover evaporates quickly and dries without residue, so the next barrier can attach securely. If you wash your skin after use be sure to rinse and dry thoroughly before the next pouch application.



Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe
Stock No 7737/7760



Spray
Spray until the skin barrier releases.



Wipes
Rub the wipe over skin as you peel off skin barrier.



Wipes
Clean off remaining tape residue.

For product questions, sampling needs, or detailed clinical questions concerning our products call:

Australia 1800 880 851

New Zealand 0800 678 669

Routine follow-up with your healthcare professional is recommended.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.

© 2019 Hollister Incorporated. AUH230. February 2020.

Australia

PO Box 375
Box Hill, VIC 3128
1800 880 851

www.hollister.com.au

New Zealand

PO Box 62-027
Mt Wellington, Auckland
0800 678 669

www.hollister.co.nz

