

ADAPT Paste

79300

79301

General Information

- Use to fill or caulk uneven skin contours near the stoma to create a flatter surface
- Use to prevent ostomy drainage from getting under the skin barrier

Using Adapt Paste

1. Remove liner on skin barrier and apply a small bead of Adapt paste at the inner edge of the skin barrier — next to the opening where the stoma will be (Figure 1). You may let it sit while you remove your used pouch.
2. Wash and dry your skin thoroughly.
3. Apply additional Adapt paste to uneven areas near the stoma if needed.
4. Apply your ostomy skin barrier as usual.

Frequently Asked Questions

Q: Does paste make my skin barrier adhere better?

A: Adapt paste prevents liquid drainage from getting between your skin and your skin barrier. This protects your skin and often extends the life of your skin barrier. Adapt paste is NOT an adhesive and too much paste may actually interfere with a good seal.

Q: What if the paste touches my stoma?

A: Adapt paste will not harm your stoma or cause it to stop draining. As you press your skin barrier into place you may see the paste filling in the area around the stoma.

Q: Do I need to let the paste “set up” before applying my pouch?

A: Some people find the paste performs better if it is allowed to sit for a short time before it is applied to the skin. If so, refer to step 1 above.

Q: How should I remove the paste?

A: The paste will come off when you remove your pouch. If you are changing your pouch after a very short time (less than a day) you may see paste residue that should be gently lifted from the skin. If a small amount of paste residue is still adhered to your skin but it appears clean, you can apply your new pouch over it.

Q: Sometimes my skin stings when I use Adapt paste, why is that?

A: Adapt paste and other ostomy pastes contain alcohol which may cause a stinging sensation if you have a skin irritation.* Adapt CeraRing barrier rings are alcohol-free and are an excellent alternative for sensitive or irritated skin.

**Irritated skin near the stoma should be evaluated by a healthcare professional who will help determine the cause of the skin problem and how to prevent further injury.*



Adapt Paste

Stock No 79300 (2.1 oz/60 g)

Stock No 79301 (0.5 oz/14 g)



Figure 1

ADAPT Paste

Additional Instructions:

For product questions, sampling needs, or detailed clinical questions concerning our products call:
Australia 1800 880 851
New Zealand 0800 678 669

Australia
 PO Box 375
 Box Hill, VIC 3128
 1800 880 851
www.hollister.com.au

New Zealand
 PO Box 62-027
 Mt Wellington, Auckland
 0800 678 669
www.hollister.co.nz

Routine follow-up with your healthcare professional is recommended.
 Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.
 The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
 © 2020 Hollister Incorporated. AUH231. February 2020.

