

REQUIRES CLINICAL AUTHORISATION

General Information

- Medical grade silicone adhesive
- Increases the adhesion of skin barriers and/or tape
- Helps to provide a secure seal

Using Medical Adhesive Spray

1. Prepare your pouching system as instructed.
2. Remove the protective release liner from the adhesive (skin barrier and/or tape) on your pouching system.
3. Shake **Adapt** medical adhesive spray can well before using.
4. Spray directly on to the adhesive side of the pouching system. Allow to dry for 5 minutes.
5. Apply your pouching system as instructed.

Frequently Asked Questions

Q: Is it necessary to use the Adapt medical adhesive spray on the entire skin barrier and tape?

A: You can spray it on the entire adhesive surface of the pouching system. However, if there are particular areas that are not adhering well, you can use it only in those areas.

Q: What is the best way to spray the adhesive?

A: Hold can so that the arrow faces the product to be sprayed. Spray in short bursts until the surface of the product is coated evenly.

Q: Can a second coat be applied?

A: Yes, if necessary. Remember to wait 5 minutes for the second coat to dry before applying your pouching system.

Q: Can I spray it directly on my skin?

A: This product should not be sprayed directly on skin and it is not necessary in order for it to be effective.

Q: Will the adhesive be difficult to remove from the skin?

A: The medical adhesive should come off easily when the pouching system is removed. If needed, the:

- Adapt Medical Adhesive Remover Spray (Product Code 7737), or
- Adapt Universal Remover Wipe (Product Code 7760)

can help with ease of removal.



Adapt Medical Adhesive Spray



Spray directly on to the adhesive, and allow to dry for 5 minutes. Then apply pouch as instructed.

