

# Hollister Non Sterile & Sterile Leg Bags

## Guide for Use

### Connecting to catheters and sheaths

1. Wash hands. Use sterile technique if attaching to catheter.
2. If using leg bag with 50cm tubing (codes 9624K, 9632K, 9652K or 9653K), cut inlet tubing to desired length if required and insert spare inlet adaptor into tubing.
3. Attach inlet adaptor to catheter or sheath.
4. Attach leg straps through the bag slits
5. Secure bag comfortably to leg with velcro closure leg straps (silicone to leg).
6. To empty leg bag, flip tap open. Snap closed to secure.
7. Always close the leg bag tap before removing a night drainage bag.

### Maintenance with Catheters

- Leave the leg bag attached to the catheter to maintain a “closed” system.
- Change the leg bag weekly using a sterile technique.

### Product Ordering Codes

#### Sterile

9621K – 500ml / 10cm inlet

9624K – 500ml / 50cm inlet

9631K – 800ml / 10cm inlet

9632K – 800ml / 50cm inlet

#### Non Sterile

9652K – 500ml / 50cm inlet

9653K – 800ml / 50cm inlet

For more information or samples please call:

Australia Customer Care **1800 880 851**

[www.hollister.com.au](http://www.hollister.com.au)

New Zealand Customer Service **0800 678 669**

[www.hollister.co.nz](http://www.hollister.co.nz)



### Cleaning the Leg Bag (for sheath drainage)

1. Remove leg bag daily.
2. Empty all urine from leg bag into toilet.
3. Close outlet tap, fill leg bag with cold tap water and shake vigorously.
4. Empty water through drainage outlet into toilet and repeat steps 3 and 4.
5. Refill bag with decontaminant solution and agitate for 30 seconds.
6. Drain solution into toilet, open outlet and hang leg bag in shaded place to air dry.
7. All parts must be completely dry before the next use. Replace leg bag weekly.

### Decontaminant Solution Preparation

**Option 1:** Household detergent solution.  
Dilute 5ml detergent in 150ml of warm tap water.

**Option 2:** Weak vinegar solution (for odour removal).  
Dilute 30ml vinegar in 120ml of cold tap water.

**Option 3:** Milton's solution

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions. © 2020 Hollister Incorporated. All rights reserved. Hollister and logo are trademarks of Hollister Incorporated. Printed in Australia. HL009. January 2020.