

InView Sheaths



Guide for Use

General Information

InView Sheaths are self-adhering and engineered with a clear, ultra fine silicone material.

Available in three styles:

- **Special:** Shorter length sheath. Adhesive Band width 50mm.
- **Standard:** Adhesive band width 40mm.
- **Extra:** 70% more adhesive than Standard length. Adhesive band width 70mm.

Selecting the Proper Size

- Five sizes are available with colour coded packaging matching the sizing guides.
- Use sizing guide to determine the correct size needed.
- Place sizing guide just behind the glans of the penis.
- If between sizes select the larger sized sheath.
- In cases of shorter shaft length, select InView Special for good adherence.
- Select the InView Extra when a larger adhesive band is needed to improve adhesion.

Cleaning Instructions on reverse.



To view a video of "Guide For Use", please visit our website

Australia: www.hollister.com.au OR

New Zealand: www.hollister.co.nz

Search with keyword "InView" and click on link to product page, scroll down for video instructions.



Sheath Measuring Guide



InView Sheaths



Guide for Use

Tips for Use

1. Clean and thoroughly dry penis. Do not use lotions, powders or ointments on the skin. Trim pubic hair as needed.
2. Remove Sheath from packaging by tearing across perforations.
3. If using a Special or Extra Sheath, first remove protective cap from the sheath. **(Fig 1 and 2)**
4. Unroll the sheath onto the penis ensuring not to push the sheath too far onto the penis. There should be a 5cm gap between the connector end of the sheath and the penis. **(Fig 3)**
5. Gently squeeze the sheath to ensure good contact between the adhesive and the skin. **(Fig 4)**
6. Moistening the connector end will give an easier and firmer fit when connecting to a bedside collector or leg bag. Ensure that there is a good fit without more than 10mm between the end of the connector and the start of the Sheath extension.
7. For easy removal use warm soap and water as this breaks down the adhesive and then gently roll the sheath off. This can be done in the shower or by using a moist towel to dampen the adhesive. Adhesive remover wipes may also be used. **(Fig 5)**

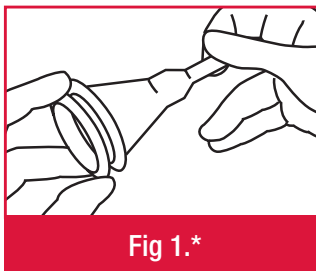


Fig 1.*

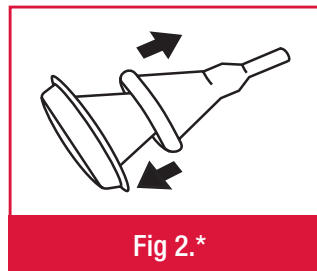


Fig 2.*

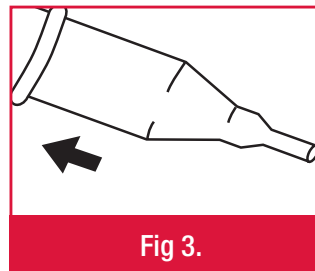


Fig 3.

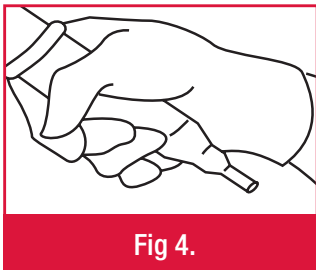


Fig 4.

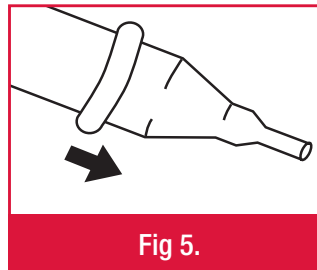


Fig 5.

* Applicable only to Special and Extra styles.

For more information or samples please call:

Australia Customer Care **1800 880 851**

www.hollister.com.au

New Zealand Customer Service **0800 678 669**

www.hollister.co.nz

Do not use on irritated or compromised skin. Do not use if allergic reaction occurs. Not made with natural latex. Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions. © 2020 Hollister Incorporated. All rights reserved. Hollister and logo are trademarks of Hollister Incorporated. Printed in Australia. HL011. January 2020.