

User Guide AnchorFast SlimFit™






Oral Endotracheal Tube Fastener

Indication and Intended Use

The oral endotracheal tube fastener is indicated for use by healthcare professionals in securing oral endotracheal tubes ranging in size from 5.0 to 8.0 mm inner diameter. The suitability of the oral endotracheal tube fastener must be assessed for each patient. (See the section below entitled "Precautions" for factors to consider.)

The Anchorfast devices do not contain any materials that are magnetic, conductive or metallic and therefore meet the requirements of ASTM F-2503 to be considered "MR Safe."




- A – Hook & loop closure (grey tab)
- B – Skin barrier pads
- C – Upper lip foam
- D – Neck strap
- E – Gliding tube shuttle on track
- F – ET tube wrap
- G – One-click security clamp

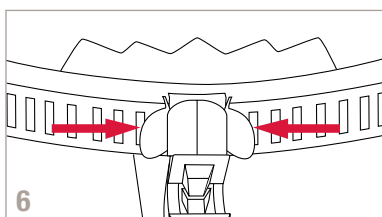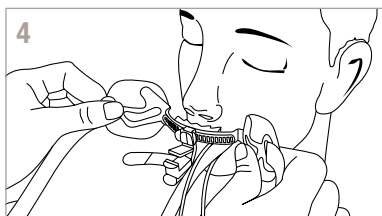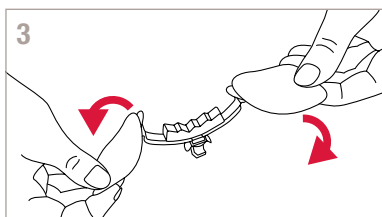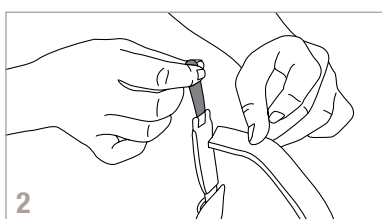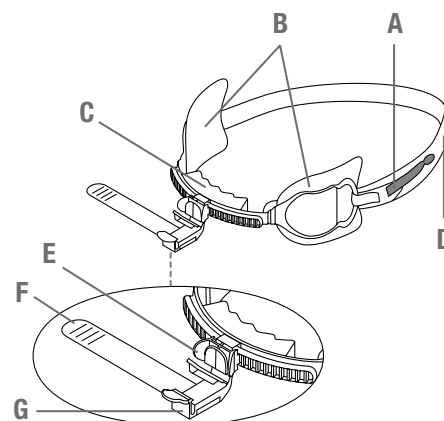

Application

Before applying the device, the patient's skin should be clean, dry, and free of oily residue. Do not use skin gel wipes or other skin preps with the oral endotracheal tube fastener.


1. Remove the product from packaging.
2. Separate hook and loop closure (grey tab) of the neck strap. Remove the release liners from the two skin barrier pads.
3. Pull back gently on the skin barrier pads so they are out of the way, to prepare for application.
4. Centre the device on the patient's upper lip, so the non-absorbent upper lip foam lightly touches the skin.
5. Press the two skin barrier pads on the patient's skin and hold in place until they adhere well. This should take approximately 30 seconds.
6. Squeeze the tabs on the sides of the gliding tube shuttle and move the clamp along the track to a location adjacent to the ET tube.
7. Remove the release liner from the "ET tube wrap" [wrap], exposing the adhesive. Before applying the wrap to the ET tube, make sure the ET tube is dry and free of any residue.
8. Position the ET tube under the non-slip grippers.
9. Loop the wrap tightly around the ET tube; avoid including any other tubes within the wrap. Pull the remaining portion of the wrap through the security clamp. Secure the wrap by snapping shut the one-click security clamp (an audible click will be heard).
10. For greater comfort and security, adjust the neck strap at the grey tab. Do not overtighten.
11. Allow two fingers width between the strap and the back of the patient's head.

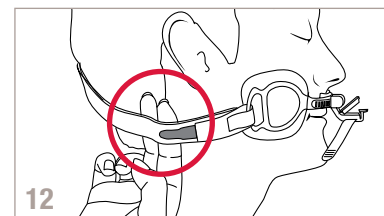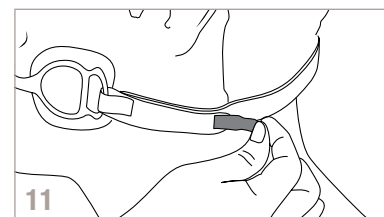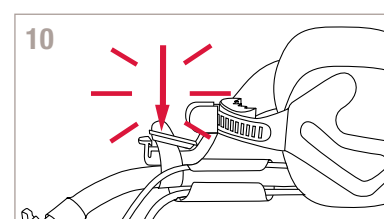

USA: Rx Only

***CAUTION:** FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR OTHER HEALTHCARE PRACTITIONER LICENSED UNDER STATE LAW TO ORDER THIS PRODUCT. PRIOR TO USING THE ANCHORFAST SLIMFIT™ ORAL ENDOTRACHEAL TUBE FASTENER, BE SURE TO READ THE PRODUCT INSTRUCTIONS FOR USE.

AnchorFast SlimFit™
Oral Endotracheal Tube Fastener

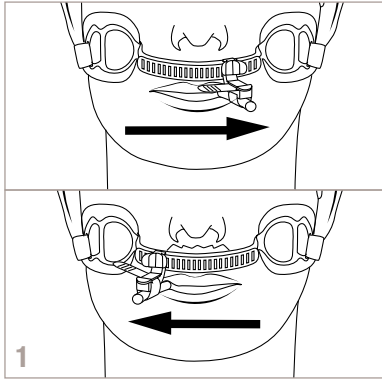

Routine Care

1. To reposition the ET tube, squeeze the gliding tube shuttle tabs on the outer edges and move in either direction along the track.
2. Reposition the ET tube from side-to-side at least every two hours or more frequently if the patient's condition dictates, to minimise the risk of injury to the skin, lips, and/or oral cavity from unrelieved pressure and shear forces.
3. Change the device when it is no longer adhering, or at least every 5 days.

Removal

1. Release the security clamp holding the wrap in place.
2. Carefully remove the wrap from around the ET tube.
3. Release the neck strap by unfastening the hook and loop closure at the grey tab.
4. Remove the skin barrier pads by gently peeling them away from the patient's skin.

Warnings

- The oral endotracheal tube fastener is indicated for single use. **To help ensure proper adhesion, do not reuse.**
- As with any fixation device, excessive pressure introduced by the device and/or patient's head and/or body position (e.g. prone or side-lying) may cause dermal injury, dental damage (associated with biting), tissue ischemia, or necrosis.
- Improper assembly and/or attachment of the device may increase the risk of hypoventilation or aspiration.

Precautions

- Store, handle, and dispose in accordance with institutional protocol and universal precautions.
- Be sure to frequently assess patient since wear time varies by patient.
- To minimise the risk of pressure injury, inspect the patient's lips, skin and oral cavity at least every two hours or more frequently if the patient's condition dictates^{1,2} (e.g. vulnerable to fluid shifts and/or oedema).
- After application of the oral endotracheal tube fastener, check the patient frequently to ensure that both the oral endotracheal tube fastener and the ET tube are secure and correctly positioned.
- Use caution in patients with full or swollen lips, facial swelling, dental appliances, restorative implants, and/or loose or protruding teeth.
- Patients without front upper teeth or unable to wear upper dentures may lack the maxillary support required to use the oral endotracheal tube fastener.
- Patients with facial hair may lack the necessary support to anchor the skin barrier pads.
- Reconfirm position, depth of intubation, and patency of the ET tube or other airway device during and after any change in the patient's head, neck, or body position, or any change in the location of the fixation device.
- Use caution during patient movement and/or repositioning to avoid dislodging the ET tube.
- To ensure proper fixation of the device, exercise caution with the use of other devices and/or instruments (i.e., feeding tubes, fibre optic (fiberoptic) scopes) within the oral cavity during endotracheal intubation.
- Discontinue use of the device if redness, skin irritation or pressure injury occurs.
- Repeated adjustment of the ET tube in a distal or proximal direction may affect the performance of the "ET tube wrap" [wrap].
- Avoid aligning the inflation lumen directly under the non-slip grippers when securing the ET tube, and including all other tubes (e.g. inflation, subglottic) within the wrap.
- Care must be taken when using endotracheal tubes with subglottic suctioning to avoid occlusion of the suction lumen.

References

1. Pieper, Barbara. "Mechanical Forces: Pressure, Shear, and Friction." Chapter 12 in *Acute & Chronic Wounds: Nursing Management*, 3rd ed., ed. Ruth A. Bryant & Denise P. Nix. (St. Louis: Mosby, Inc., 2007).
2. Panel for the Prediction and Prevention of Pressure Ulcers in Adults. "Pressure Ulcers in Adults: Prediction and Prevention." *Clinical Practice Guidelines*, Number 3. AHCPR Publication No. 92-0047. (Rockville, MD: Agency for Health Care Policy and Research, Public Health Service, U.S. Department of Health and Human Services. May 1992.)

In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.Hollister.com/authority or contact EC Rep or local distributor.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions. Not all products are CE marked.

Australia

PO Box 375
Box Hill VIC 3128
1800 880 840

www.hollister.com.au

For more information, visit
www.Hollister.com.au/AnchorFast

New Zealand

PO Box 107097, Auckland
Airport, Auckland 2150
0800 678 669

www.hollister.co.nz

For more information, visit
www.Hollister.co.nz/AnchorFast